



Maternity bed

# FREYA 3




## Maternity bed

# FREYA 3

A modern childbirth bed constructed of 4 segments (the seat segment is fixed), supported by 3 columns, electrically controlled.

The product enables the adjustment of positions by changing the height of the sleep surface, inclination of the back plate and the lower thighs rest, longitudinal inclinations (Trendelenburg and anti-Trendelenburg), manoeuvring and the use of the CPR function for rapid lowering of the back plate. The bed can be controlled by means of one of the three systems: central panel (supervisor), remote control cable or control keyboards placed in the rails. In order to increase the patient's security, the control panels which she can reach by the hand (on the inside of the rails) have the functions of a lamp and calling the medical staff. For similar reasons, the operation of the remote control requires authentication (a special key). Panels on the outer side of the rails enable fast setting of the Trendelenburg or reverse Trendelenburg position. The central control panel enables immediate setting of classic positions (Fowler, „zero”, examination, „cardiologic”). Possibility to install an arm holder and drip-bag hanger on the bed's frame.

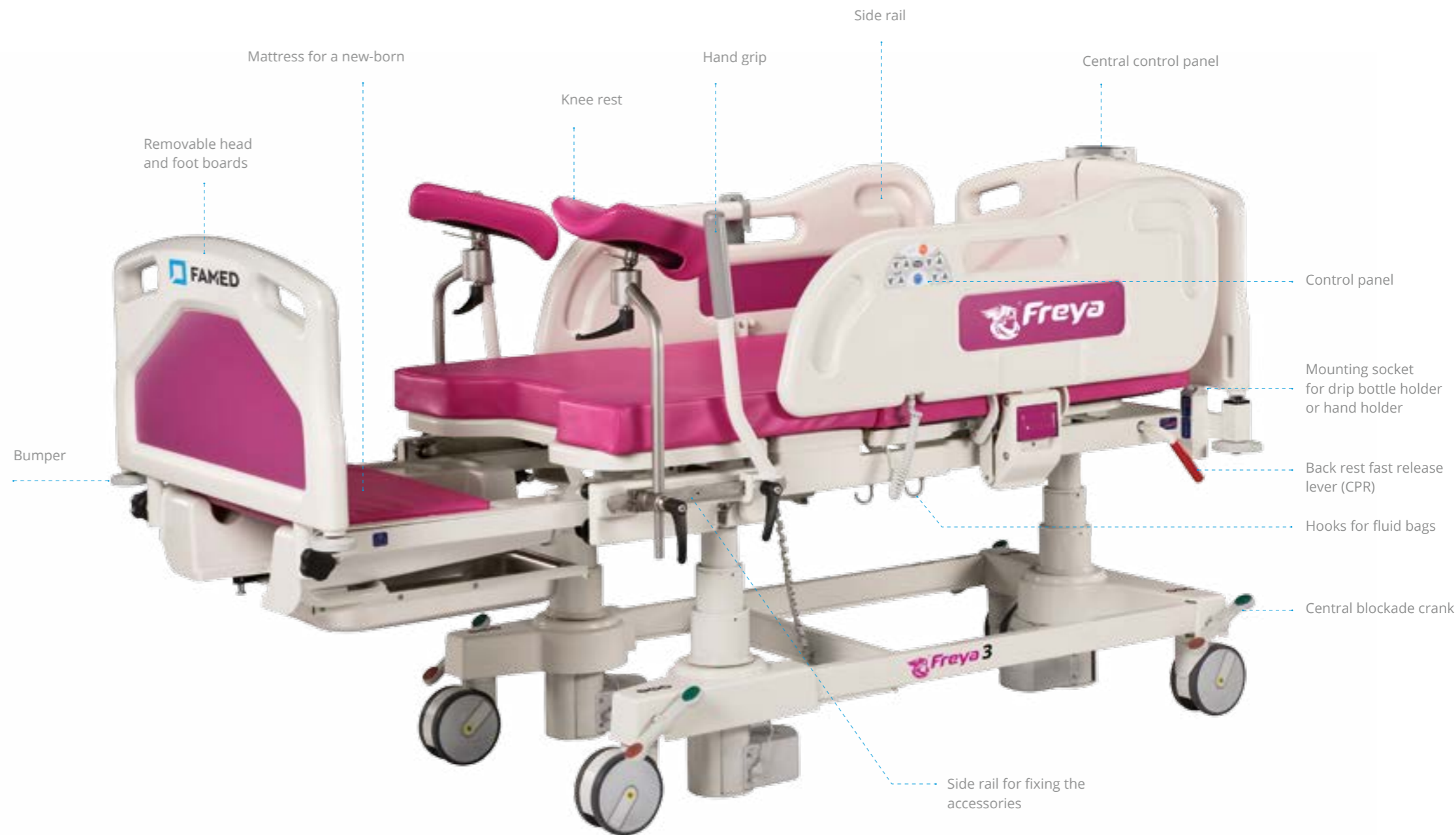

### Standard equipment:



Mattress for a new-born



Easily removable fluid bowl



Central control panel (supervisor)



Back rest fast release lever (CPR)



Foot rests



Knee rests



### Advantages of the Freya bed:

- Wide range of possible position adjustments
- 3 control systems (including one for the patient)
- Low bed surface height, making it easier to get up
- Very good technical parameters



Proctoscope examinations



Horizontal position



Horizontal position with a mattress for a new-born



Reclining position

### Selected positions of the Freya bed:

- The bed can easily be converted into an armchair, ensuring a comfortable position for the woman in labor.
- Cardiology chair's position setting
- Examination position setting
- Fowler position setting
- Zero position setting



Gynaecological examinations



Sitting position



Sitting position



### Electric function control using:

- Central control panel (supervisor)
- Wired manual remote with lock (key)
- Control panels in side rails from the outer side with the Trendelenburg and reverse Trendelenburg positions
- Control panels in side rails from the inner side with a function of night-time floor illumination and medical personnel calling
- Foot controller (optionally)



### Side rails

equipped with a folding mechanism



Reverse Trendelenburg position



Trendelenburg position

### Utility parameters:

- Bed mechanisms allowing for bed surface height adjustment, back and leg rest angle position adjustment, Trendelenburg and reverse Trendelenburg positions and rollover.
- Ability to adjust the back rest angle position using the CPR handle.
- Bed surface height, longitudinal lean (Trendelenburg and reverse Trendelenburg positions), back and leg rest (thigh segment) angle adjustment using electric actuators.

### Control panels



Central control panel (supervisor)



Central control panel and remote controller



Control panel in side rail

### Available head and foot boards



Plastic head and foot boards with inserts



RAL 5002 RAL 5024 RAL 6019 RAL 4010



RAL 6027 RAL 1015 RAL 1018

Available colours of inserts for head and foot boards and side rails

## Technical data

Total length	2165 mm ± 20 mm
Total length in chair position	1780 mm ± 20 mm
Total width (with side rails)	965 mm ± 5 mm
Minimum height of the bed frame	580 ± 10 mm
Maximum height of the bed frame	980 ± 10 mm
Seat section inclination angle	30° ± 5°
Back rest inclination angle	70° ± 5°
Trendelenburg position	20° ± 2°
Reverse Trendelenburg position	20° ± 2°
Total table weight	about 160 kg
Permissible load	250 kg
Permissible load of withdrawable segment	170 kg
Power supply	230 V~ (optional 100 V~, 120 V~), 50/60 Hz
Power consumption	max 250 VA
Class of protection	II
Type part of the application	B
Degree of protection	at least IP-X4
Period of use	10 years



**Famed Żywiec Sp. z o.o.**  
ul. Fabryczna 1  
34-300 Żywiec, Poland

**Head Office:**  
tel.: +48 33 866 62 00  
fax: +48 33 475 58 90

**Export Department:**  
tel.: +48 33 866 62 58  
tel.: +48 33 866 62 65  
tel.: +48 33 866 62 66  
fax: +48 33 861 46 78

expo.famed@famed.com.pl  
www.famed.com.pl



The product meets the requirements of the European Directive MDD 93/42/EEC concerning safety of medical devices

Edition 05/2015/01. Famed Żywiec reserves the right to make modifications in the products in the course of technical progress. All illustrations and photos, used in this material, are for illustration and may not accurately depict the actual product.

